



Recent Infection Results Documentation Form

Rapid Test for Recent Infection									
Test Name: Asanté or Maxim Swift		Sample Type (circle): Finger prick Venous blood Plasma Serum							
Kit Lot#		Kit Expiry Date		Site/Facility Name		Tester Name			
No	Test Date	Participant ID	Visual Results (Mark "√" if line is present)			Recency Interpretation [All 3 lines = LT; C & V lines = Recent only C line = Neg] (Please circle one)			
			Control line	Verification Line	Long Term (LT) Line				
		QC - Long Term				LT	Recent	Neg	Invalid
		QC - Recent				LT	Recent	Neg	Invalid
		QC - Negative				LT	Recent	Neg	Invalid
1						LT	Recent	Neg	Invalid
2						LT	Recent	Neg	Invalid

TO SET UP FORM:

- Specify type of test next to "Test Name" on the form: Asanté or Maxim Swift.
- **Specify sample type**
- Document the kit lot number under "Kit Lot #" field and the kit expiration date under the "Kit Expiration Date" field.
- **Insert facility name under the "Site/Facility Name" field, and the name of the person who conducted the test under "Tester Name" field.**

FOR EACH SPECIMEN:

- Insert the date of specimen collection under "Test Date" column.
- **Insert Participant ID Number.**
- Under "Visual Results," make with a check mark if control line, verification line, and/or long term (LT) line is present.

Interpretation

- If 3 lines are present, circle "LT"
- If the control and verification lines are present, circle "Recent"
- If only the control line is present, circle "Neg"